

Reconnecting the Remote (Android)

Note: Android phones may have slight variations in controls and layout.




If your previously paired remote is having trouble connecting to the Inspire Sleep app, follow this guide to get re-connected.

- 1 From the Menu in the Inspire Sleep App, tap "My devices"




- 2 Tap "Forget device"

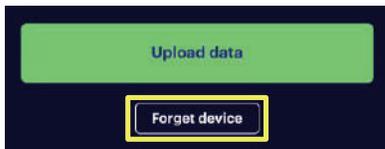

- 3 Tap "Go to Settings" to open your phone's Bluetooth Settings

- 4 If device is still listed, tap gear icon
If device is not listed, skip to Step 6.

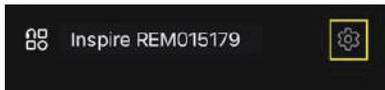


- 5 Select "Unpair" or "Forget"
Depending on Android type, you may see the option to select "Forget" instead of "Unpair".




- 6 Press and hold blue button on remote for 8 seconds



- 7 Open the Inspire Sleep App and navigate to "My devices" from the Menu

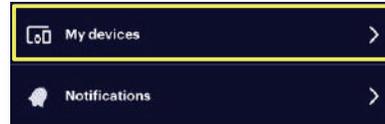


- 8 At the bottom of the screen, tap "Add a new device"

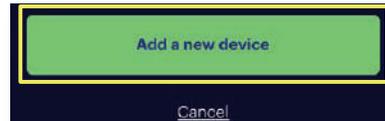


- 9 Ready Remote for Pairing

- Press OFF button on the remote and hold to chest.
- Listen for the tones.
- Then, press the Bluetooth button and hold for 3 seconds and continue app instructions.

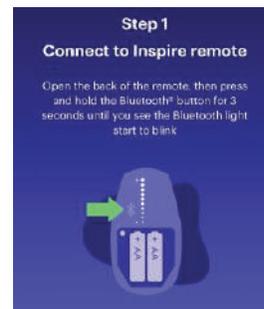

- 10 Pair Remote to Phone

- Tap "PAIR" to pair the remote to the phone's Bluetooth.
- Then lock the connection by pressing and releasing the blue button in the 2580 remote.

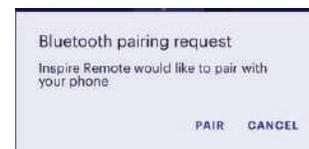


Reconnecting the Remote (Android)



⚠️ Deleting the App for Android.

- 1 Follow Steps 1–5 to Forget Device
- 2 Turn off Bluetooth
- 3 Delete the Inspire Sleep App
 - Go to Settings, then tap Apps.
 - Under Recent, tap See All Apps.
 - Tap the Inspire Sleep App and then tap Uninstall.

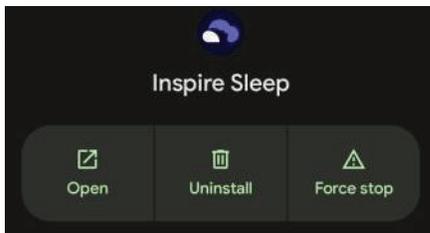

- 4 Remove Batteries from Remote


Open back of remote and remove both AA batteries.



- 5 Turn Phone Off
 - Tap volume up and power at the same time, tap Power Off.
 - Wait 60 seconds.
- 6 Turn Phone Back On

Hold down power button and allow the phone to reboot.
- 7 Toggle Bluetooth Back On

- 8 Go to Play Store and Reinstall App
 - Open Play Store and search “Inspire Sleep” to find the app.
 - Tap the Install button to reinstall the app.

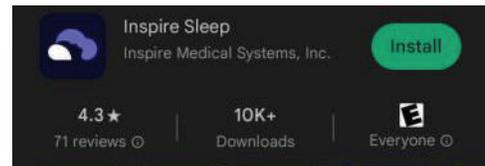

- 9 Reinstall Remote Batteries

Put both batteries back into the remote and replace the cover.



- 10 Open the App and Sign In
- 11 Upload Data

Follow the New Device Pairing prompt and go through the pairing process.

Contact Patient and Physicians Services for further assistance: **844-672-6720**